







# SIMICON-RI

## Assessment-Aid

Result	Assessment	Action needed
	Reference indicator: Spot of test soil completely visible	
	Insufficient cleaning process:	
	Spot of test soil completely visible	Do not release batch!  Check load, temperature, dosage and time.  Repeat test.  In case of repeated insufficient cleaning process: Call service technician.
	Spot of test soil visible	
	Spot of test soil visible	
	Spot of test soil slightly visible	
	Successful cleaning process:	
	Spot of test soil completely removed	Release batch. No further action needed.